



This Quick Start Guide gives simplified set-up instructions for your Henson 9000.

Please refer to the Henson 9000 User Manual for detailed set-up and operating instructions before you begin using this device.



# Henson 9000

Visual field analyzer

## QUICK START GUIDE

Please read before taking the product out of the box



### Accessories kit content includes:

- Patient response button
- Occluder
- Dust cover
- Power cable
- Dual USB cable
- USB flash drive
- Trial lens arm
- Response button holder

**Product model:** Henson 9000  
Medical Device Class 1



Specifications are subject to change without notice.

[topconhealthcare.com](http://topconhealthcare.com)

©2021 Topconhealthcare MCA#4452

# UNAPPROVED

**TOPCON** Healthcare  
SEEING EYE HEALTH DIFFERENTLY

## Product handling and set-up

### 1 Take the Henson 9000 out of the box

Always lift and move the device from the base. Take care when lifting.



### 2 Place the Henson 9000 into the location for use

- The Henson should be placed on a flat surface, with no part of the unit overhanging the edge of the surface.
- This equipment is not suitable for use in an oxygen rich environment or operation in environments where handling of fluids is normal use.
- The patient aperture should be positioned away from direct light.

NOTE: Do not connect the dual USB cable to the computer until after you have installed the software.

### 3 Set up the Henson 9000

- Connect the supplied dual USB cable, the patient response button and the power cable to the Henson 9000.
- Ensure the connected leads do not trail on the floor and are not subject to abrasion on sharp edges.
- Access to the mains supply plug and socket must be maintained at all times as this is the means of isolating the device from the mains supply.



### 4 Switch on the Henson 9000



### 5 Note the license key information

Make a note of the 25-digit license key number on the back of the machine which will be required to license your Henson 9000 software, e.g. 12345-67890-12345 67890-12345.



# UNAPPROVED

## Software installation

### Install the software before plugging the dual USB cable into the computer.

- Only computers and peripherals that comply to EN 60950 or IEC 62368-1 should be used.
- A Windows® Professional PC\* is required to operate this software. Computer and network security is the responsibility of the user.

### 1 Insert the supplied USB flash drive into a PC\*

- Remove all USB devices from the computer.
- Insert the supplied USB flash drive into the computer and wait for the prompt.
- Select 'Open folder to view files'.
- Open the folder called 'Software' and right-click on the software EXE file.
- Select "Run as administrator".
- Follow the on-screen instructions to install the software.



### 2 Plug in the USB cables


- Once the software is installed, plug in the dual USB cable between the Henson 9000 and the computer.
- Also plug in the USB device you will use for database backups (if required).



### 3

### Open the Henson 9000 Options program


- Double click the 'Henson Options' icon on the Desktop. Follow the configuration wizard to set up your software.
- On the 'Address' tab fill in your practice details.
- On the 'Backup' tab make sure that the USB flash drive letter is selected.
- On the 'Video' tab ensure there is a message saying "Defaults have been downloaded from the Henson 9000."
- If a built in webcam picture is displayed, select the Henson camera from the drop-down device list.
- On the 'Registration' tab enter the 25-digit license key number from the rear of the Henson 9000.
- Click the 'Computer' tab and check there is a device listed.
- Set any other options then click 'Save'.

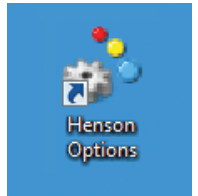

### 4

### Open the Henson 9000 software

Double click the 'Henson 9000' icon on the Desktop. (To make the Henson 9000 software start up automatically when the computer is started add it to your Startup folder.) You will now see the Henson 9000 main menu.

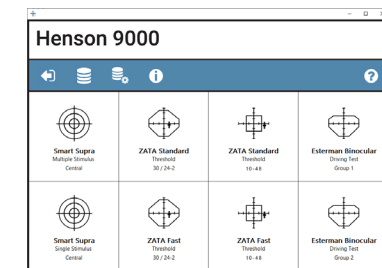

**FOR MORE INFORMATION**  
[topconhealthcare.com](http://topconhealthcare.com)

\* System requirements  
MS Windows® Professional, v.8, and above.  
Microsoft® and Windows® are either registered trademarks or trademarks of Microsoft Corporation in the United States and/or other countries.

# UNAPPROVED