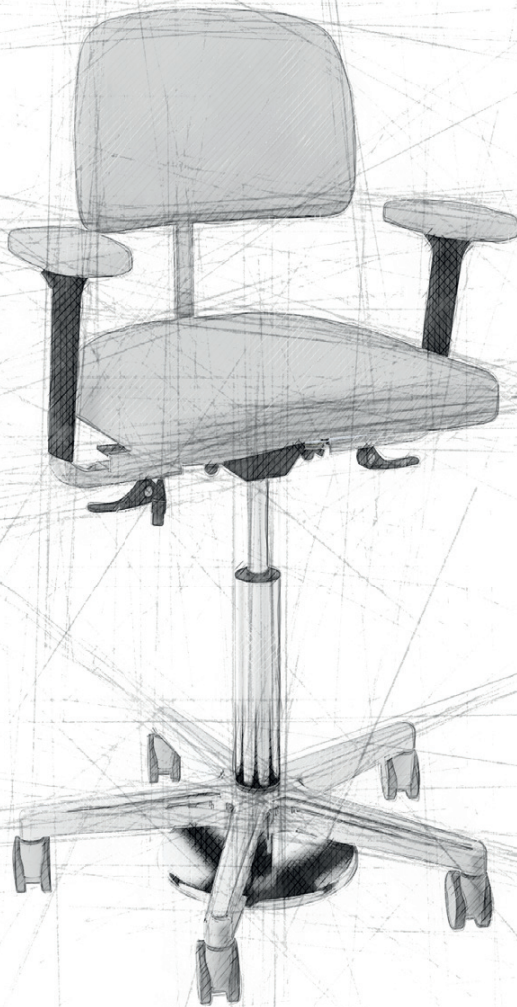
BEDIENUNGSANLEITUNG
USER GUIDE

BALANCE SUPREME

BRUMABA
OPERATING TABLE SYSTEMS

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BALANCE SUPREME

INHALT

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
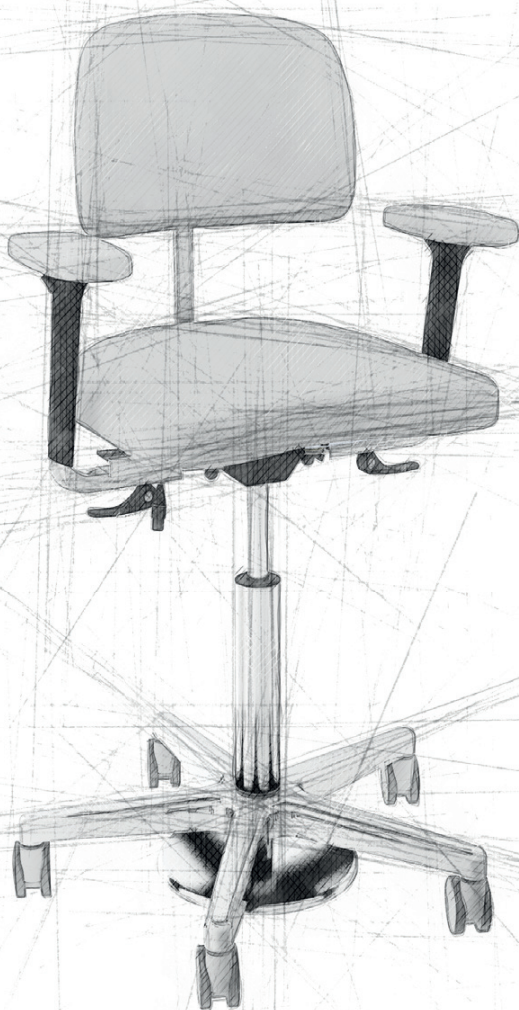
BALANCE SUPREME

CONTENT

HEIGHT ADJUSTMENT	 18
TILTING THE BACKREST	 19
TILTING OF THE SEAT	 20
BACKREST HEIGHT ADJUSTMENT	 21
ARMRESTS HEIGHT ADJUSTMENT	 22
ARMREST LENGTH ADJUSTMENT	 23
ARMRESTS ROTATION	 24
ARMRESTS PADDING ROTATION	 25
TECHNICAL DATA	 27

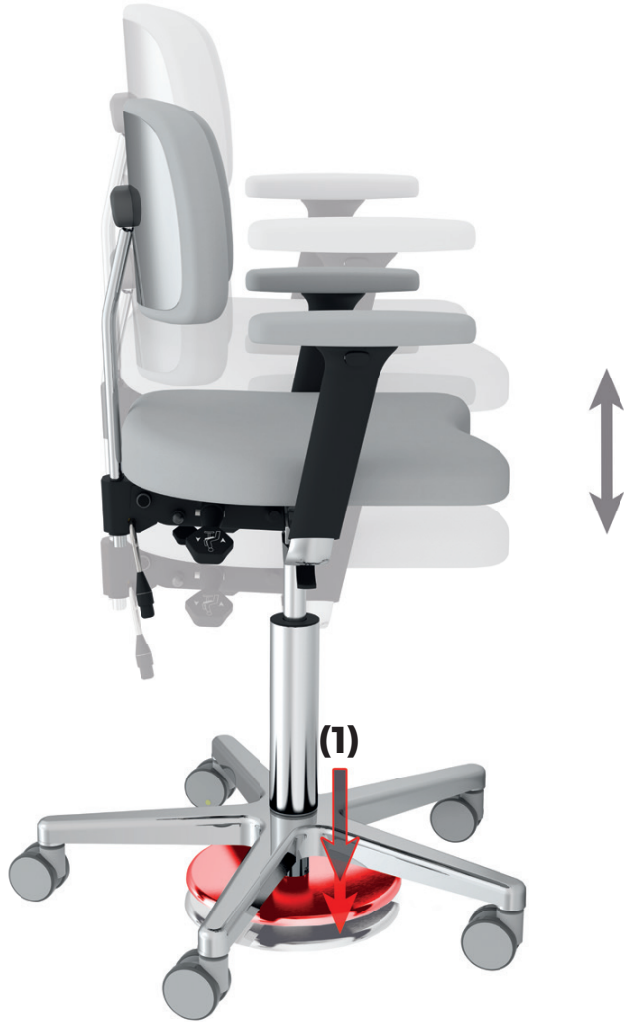HEIGHT ADJUSTMENT

- Press the trigger (1) with your foot
- Relieve the seat to move it upwards
- Strain the seat to move it down
- Take your foot off the trigger

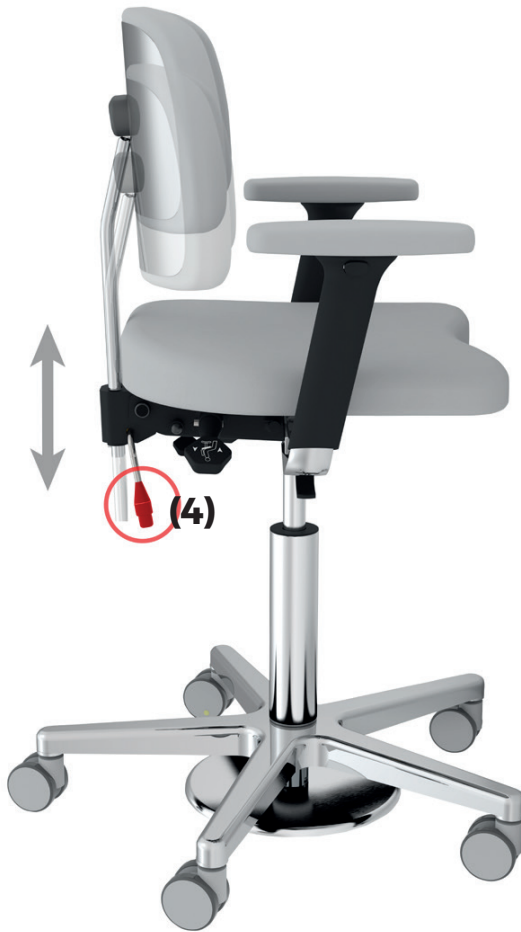
TILTING THE BACKREST

- Pull the lever (2) on the left side of the chair
- Lean forward or backward to move to the desired inclination
- To lock, release the lever

TILTING OF THE SEAT

- Pull the lever (3) on the right side of the chair
- Adjust the seat angle by leaning backwards or forwards
- Release the lever

BACKREST HEIGHT ADJUSTMENT

- Turn the lever (4) backwards to release the fixation
- Set the desired height
- Close the lever again

ARMRESTS HEIGHT ADJUSTMENT

- Press the side button (5) to adjust the height

ARMRESTS LENGTH ADJUSTMENT

- Move the self-fixing armrest

ARMRESTS ROTATION

- Swing the armrest forwards or backwards

ARMRESTS PADDING ROTATION

- Turn the self-fixing armrest padding

TECHNICAL DATA

5 ball bearing wheels made of polyurethane,	
1 x conductive	Ø 65 mm
Height adjustment	480 - 680 mm
Backrest tilt	+6° / -16°
Tilting area backrest cushion	25°
Height adjustment backrest	100 mm
Inclination seat surface	+5° / -8°
Dimensions of seat (W x H x D)	460 x 60 x 420 mm
Maximum load	120 kg
Weight	18,2 kg

The complete conductivity of the rollers and the chair is only given at black-conductive upholstery color.

CARE

Periodically remove dirt and disinfectant residues from all surfaces with a mild, commercial detergent.

A wipe disinfection is recommended. Use tested and approved agents only. Do not use agents containing the active ingredients phenol, peracetic acid and other oxygen scavengers or sodium hydrochloride and iodine scavengers.

WARNING

Check screw connections regularly and tighten if necessary!

Check the clamping effect of the armrest clamping levers before use!

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BRUMABA GmbH & Co. KG